

Sterile Hydrocolloid Wound Dressing

Introduction


The Hydrocolloid dressing being a family of wound management products contains gel forming agents like Carboxymethylcellulose Sodium etc. They are combined with elastomers and adhesives over a sheet of polyurethane film to form an absorbent, self-adhesive water proof dressing. In the presence of wound exudates the adhesive mass absorbs liquid and forms a gel, the properties of which are determined by the nature of the formulation, some dressing form a cohesive gel, which is largely contained within the adhesive matrix: other form more mobile, less viscous gels which are not retained within the dressing structure.

Indications

- ✓ Low to moderately exudating wounds
- ✓ Pressure ulcers
- ✓ Infected wounds

Application

- Assess, debride and clean the wound according to the protocols you or your facility would use
- Treat with medication if infected
- Apply the **Hycoloid™** over the entire wound
- Secure the primary dressing suitably
- Change the Dressing as per the condition of the wound and soakage - normally once in three days

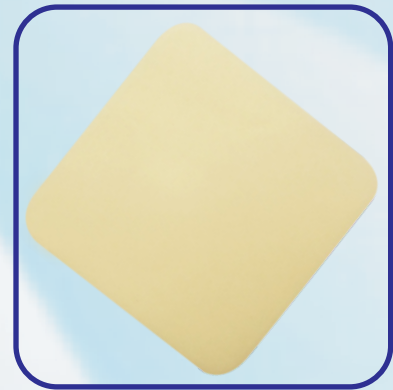

Storage

Hycoloid™ pack can be stored at normal room temperature.

Availability

Available in Gamma Sterilized sterile packaging with and without adhesive borders as per the following details.

Ordering Information - (a) Hycoloid™ - Without Adhesive Border

CODE	DESCRIPTION	UNITS
HCDP 1001-05	Hycoloid™ , 5cm x 5cm	Box of 5
HCDP 1002-05	Hycoloid™ , 10cm x 10cm	Box of 5
HCDP 1003-05	Hycoloid™ , 10cm x 20cm	Box of 5
HCDP 1004-05	Hycoloid™ , 15cm x 15cm	Box of 5
HCDP 1005-05	Hycoloid™ , 20cm x 20cm	Box of 5
HCDP 1006-05	Hycoloid™ , 10cm x 15cm	Box of 5

Ordering Information - (b) Hycoloid™ - With Adhesive Border

CODE	DESCRIPTION	UNITS
HCDA 1001-05	Hycoloid™ , 5cm x 5cm	Box of 5
HCDA 1002-05	Hycoloid™ , 10cm x 10cm	Box of 5
HCDA 1003-05	Hycoloid™ , 10cm x 20cm	Box of 5
HCDA 1004-05	Hycoloid™ , 15cm x 15cm	Box of 5
HCDA 1005-05	Hycoloid™ , 20cm x 20cm	Box of 5
HCDA 1008-05	Hycoloid™ , 10cm x 15cm	Box of 5



Manufactured by:

EUCARE PHARMACEUTICALS PRIVATE LIMITED

Plot No. AC 25-B, SIDCO Industrial Estate, Thirumudivakkam, Chennai - 600 044, India.
Tel: +91-44-4598 9955 / 9909 / 9904 / 9908 / 9903 / 9915, Fax: +91-44-2478 2516 / 4598 9914
E-mail: eucare@vsnl.com, Website : www.eucare.in / www.eucareindia.com

HCLA2-0613

We create and surpass our own standards

Hycoloid™ - Trade Mark under Registration in India.

EUCARE
the collagen people