

Köllagen-M[®]

Köllagen[®]

NeuSkin[™]

The Gold Standard in Burns Management

Most Proven concept for more than a Decade

Köllagen[®]



STERILE COLLAGEN SHEETS

In Wet Form (Bovine Origin)

Available in a sterile preserving medium, ready for use in Burns & Wounds;
Gamma Sterilized.

Sizes :
5x5cm, 10x10cm, 10x20cm, 10x25cm, 15x30cm & 20x40cm



Köllagen-M[®]



STERILE MESHED COLLAGEN SHEETS

In Wet Form (Bovine Origin)

The idea of meshing is not to effect expansion of the sheets but as a choice to treating surgeons for usage on exudating/infected Burns & Wounds;
Gamma Sterilized.

Indications & Precautions similar to Köllagen[®].

Sizes :
5x5cm, 10x10cm, 10x20cm, 10x25cm, 15x30cm & 20x40cm



NeuSkin[™]



STERILE COLLAGEN SHEETS

In Dry Form (Bovine Origin)

Ready for usage in Burns & Wounds; No pre-treatment with normal saline is required but the wound bed is to be wet with normal saline before application.
Gamma Sterilized.

Indications & Precautions similar to Köllagen[®].

Sizes :
5x5cm, 10x10cm, 10x20cm & 15x30cm



PHOTOGRAPHIC RECORD OF BENEFITS OF Köllagen



(1) Pre Köllagen



(2) Köllagen Applied



(1) Pre Köllagen



(2) Köllagen Applied



(1) Pre Köllagen



(2) Köllagen Applied



(3) Post Köllagen



(4) Healed



(3) Post Köllagen



(4) Healed



(3) & (4) Healed



EUCARE
the collagen people

Köllagen®

Köllagen-M®

NeuSkin™

INDICATIONS FOR USE

- ✓ 2nd degree superficial and deep dermal burns
- ✓ 3rd degree burns as a temporary cover after Escharectomy / Tangential excision
- ✓ Skin Donor Sites
- ✓ Traumatic loss of skin cover
- ✓ Temporary wound cover in major open fractures before definitive flap cover
- ✓ Chronic skin ulcers
- ✓ Shallow pressure sores
- ✓ Leprosy ulcers
- ✓ Dermabrasion areas
- ✓ Protective cover over widely meshed autografts

SALIENT FEATURES

- ☐ Single application on clean wounds
- ☐ Breathes like skin
- ☐ Effective barrier against infection
- ☐ Immediate pain relief
- ☐ Quick and firm adherence
- ☐ Promotes epithelialization
- ☐ Protects regenerating epithelium
- ☐ Preserves the local heat
- ☐ Stimulates healthy granulation
- ☐ Rapid and effective wound healing
- ☐ Non-Toxic & Non-immunogenic
- ☐ Biocompatible as per EN ISO 10993 Standards

DIRECTIONS FOR USE

- ☐ Denuded areas should be thoroughly cleaned of external contamination
- ☐ Infected wound should be thoroughly debrided
- ☐ Select the right dimension of Collagen sheet for application
- ☐ Squeeze out the preserving medium from **Köllagen® / Köllagen-M®** sheet and then rinse in normal saline before application
- ☐ Apply the sheet firmly on the raw area and ensure that the air bubbles are removed for an effective adhesion
- ☐ Option of a secondary dressing is left to the doctor's choice. A light gauze dressing over a non-adherent tulle may be opted.
- ☐ Collagen sheet peels off as the wound heals & epithelialization occurs; one can place a saline moist pad over Collagen dressing for few minutes to facilitate easy removal
- ☐ For Chronic Skin Ulcers : 2 to 3 applications of Collagen dressing in the interval of about 4 days after a thorough cleaning each time enhances the healing process

PRECAUTIONS & WARNINGS

- ☐ Grossly infected wound may reject the collagen cover
- ☐ On application of Collagen Sheet on infected sites, care should be taken to inspect the wound condition frequently and change of dressing is recommended after cleansing with normal saline
- ☐ Avoid in patients hypersensitive to Collagen
- ☐ The product will not be sterile when the packing is damaged during transit
- ☐ Store at normal room temperatures

SAVINGS.....



- ☐ Savings on huge dressings, IV fluids, Proteins
- ☐ Savings on Analgesics
- ☐ Savings on Antibiotics because complications like septicemia is avoided
- ☐ Savings on time & energy for Doctors and Nursing staff
- ☐ Reduces Hospital stay
- ☐ Overall cost effective & most economical

Köllagen® - Ordering Information

CODE	DESCRIPTION	UNITS
KGNS 1001-05	Köllagen® , 5x5cm	Box of 5
KGNS 1005-05	Köllagen® , 10x10cm	Box of 5
KGNS 1007-02	Köllagen® , 10x20cm	Box of 2
KGNS 1008-02	Köllagen® , 10x25cm	Box of 2
KGNS 1011-02	Köllagen® , 15x30cm	Box of 2
KGNS 1014-02	Köllagen® , 20x40cm	Box of 2

NeuSkin™ - Ordering Information

CODE	DESCRIPTION	UNITS
NSND 1001-10	NeuSkin™ , 5x5cm	Box of 10
NSND 1002-05	NeuSkin™ , 10x10cm	Box of 5
NSND 1004-05	NeuSkin™ , 10x20cm	Box of 5
NSND 1006-05	NeuSkin™ , 15x30cm	Box of 5

Köllagen-M® - Ordering Information

CODE	DESCRIPTION	UNITS
KGMS 1001-05	Köllagen-M® , 5x5cm	Box of 5
KGMS 1002-05	Köllagen-M® , 10x10cm	Box of 5
KGMS 1003-02	Köllagen-M® , 10x20cm	Box of 2
KGMS 1004-02	Köllagen-M® , 10x25cm	Box of 2
KGMS 1005-02	Köllagen-M® , 15x30cm	Box of 2
KGMS 1006-02	Köllagen-M® , 20x40cm	Box of 2



Manufactured by:

EUCARE PHARMACEUTICALS PRIVATE LIMITED

Plot No. AC 25-B, SIDCO Industrial Estate, Thirumudivakkam, Chennai - 600 044, India.

Tel: +91-44-4598 9955 / 9909 / 9904 / 9908 / 9903 / 9915, Fax: +91-44-2478 2516 / 4598 9914

E-mail: eucare@vsnl.com, Website : www.eucare.in / www.eucareindia.com

We create and surpass our own standards

Köllagen® - Registered Trade Mark in India. **Köllagen-M®** - Registered Trade Mark in India. **NeuSkin™** - Trade Mark under Registration in India.